


ARCHOS Connected Self App

[Blood Pressure Monitor] Details on the application


To access details on the Blood Pressure Monitor simply touch the green banner in the main menu.

ARCHOS Connected Self App

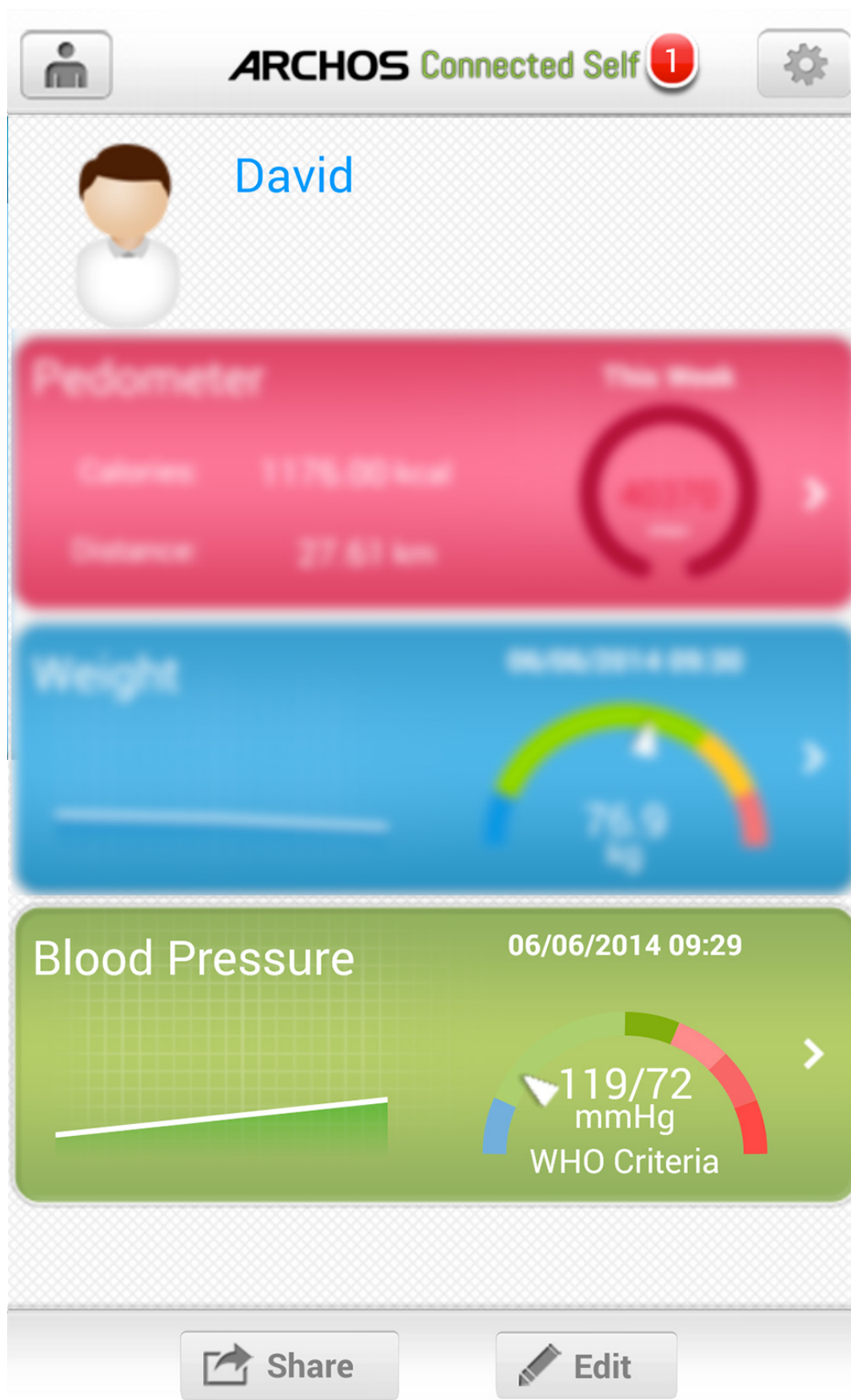

In the new screen that opens you can consult the history of your measurements.

ARCHOS Connected Self App

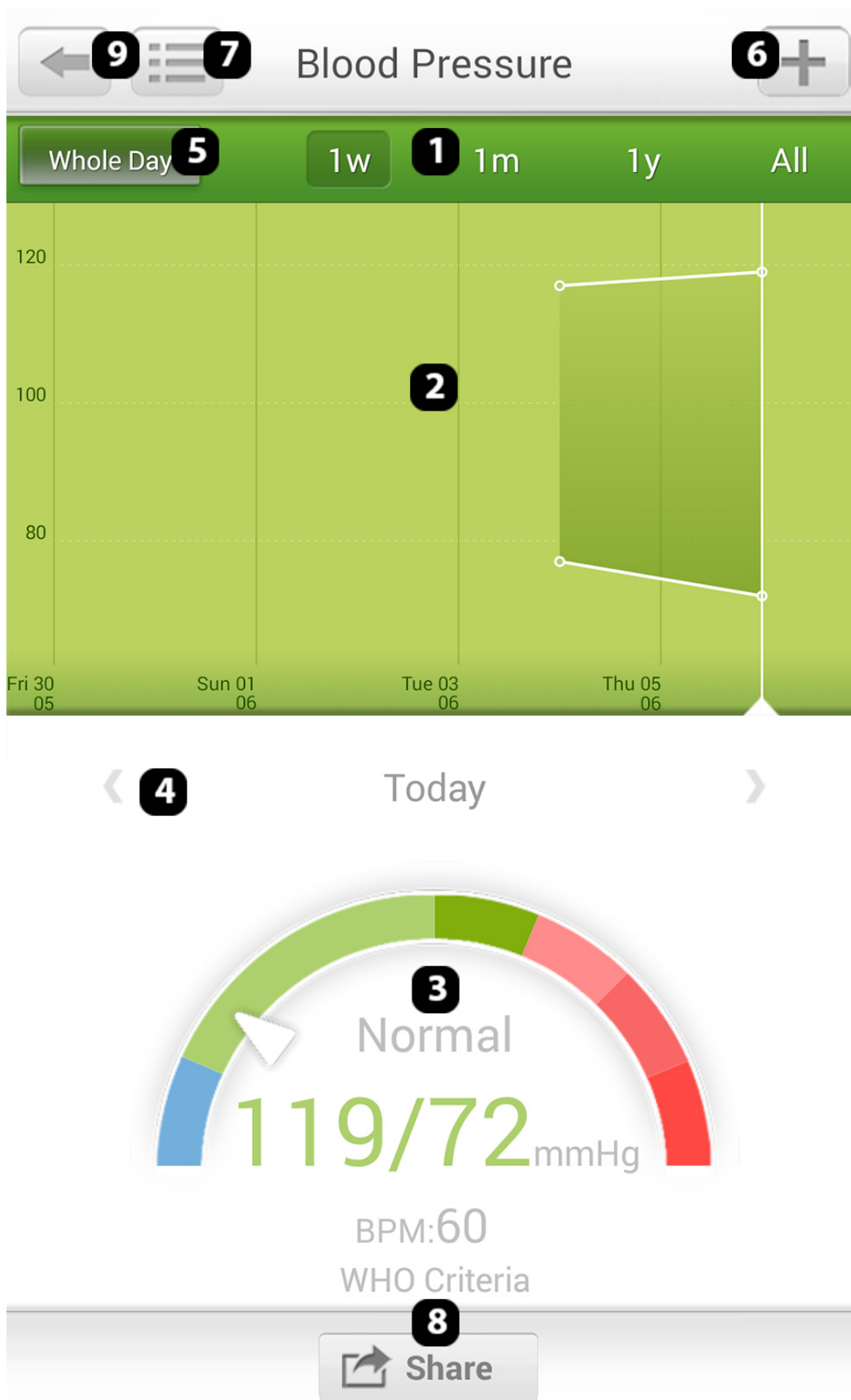

ARCHOS Connected Self App

1. Choose the display type for the graph: by week / month / year / total

2. Progression graph

3. Complete measurement data showing: arterial pressure (Systolic and diastolic) , beats per min, how your data compares to world health guidelines

4. Lets you check a measure on a specific date

5. Lets you change the data displayed to data recorded in the morning / day / evening or over 24hrs

6. touch to manually add a measurement


7. lets you change the data displayed)

8. Share your measurement on Facebook, Twitter or e-mail

9. Go back to the previous menu

If you want to change how your measurements are displayed touch .

ARCHOS Connected Self App

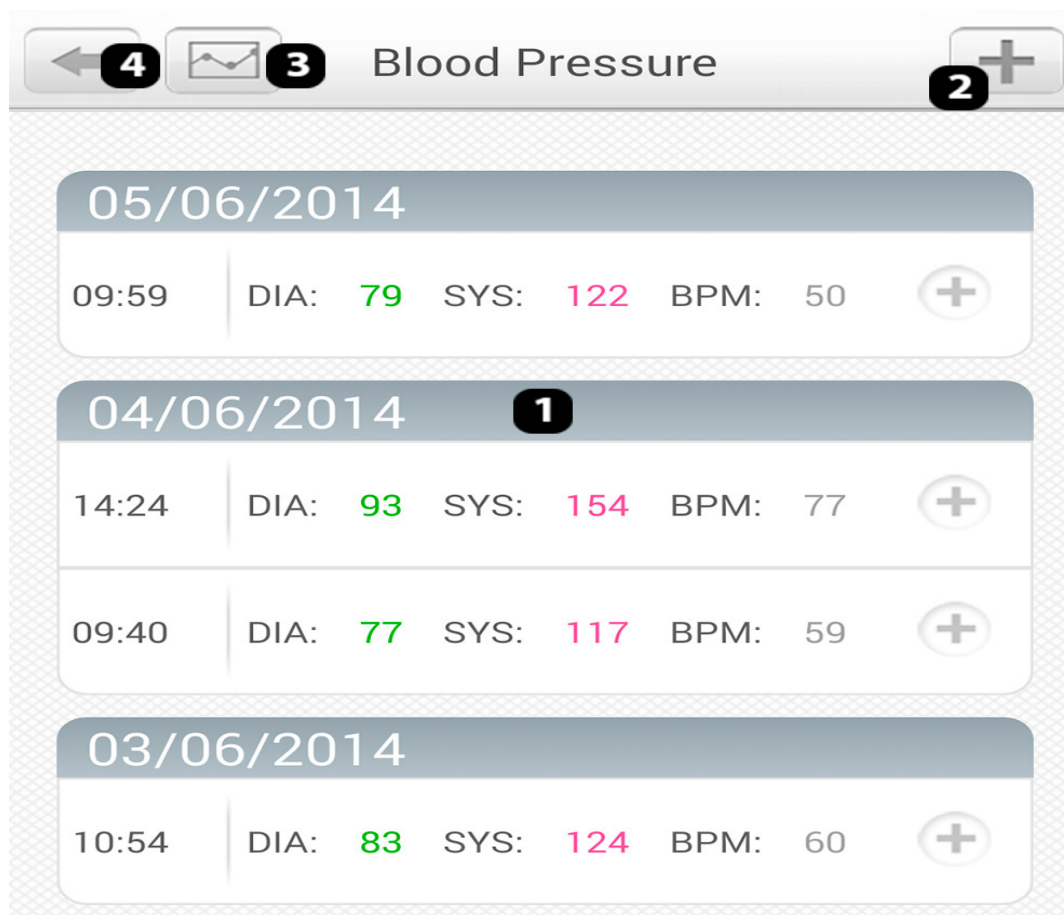

1. List of synced measurements

2. Touch to manually add a measurement

3. Go back to the graph display

4. Go back to the previous menu

Unique solution ID: #1336

Author: Admin FAQ

Last update: 2014-06-25 14:44